Transcend AUTO™ Quick Guide

To view Transcend Instructional videos
go to www.mytranscend.com/support
or scan with QR code reader:

ENGLISH

FRANÇAIS

DEUTSCH

ITALIANO

ESPAÑOL

NEDERLANDS

PORTUGUÊS

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TRANSCEND INTRODUCTION

Thank you for choosing Transcend AUTO, the first auto-titrating obstructive sleep apnea therapy device designed to fit your active lifestyle. Transcend Auto is a fully-featured APAP $R_{x_{only}}$ that continually monitors your breathing to adapt to your changing therapy needs. This device provides pressure relief on exhalation with EZEX pressure relief technology. Weighing less than a pound (426 g) Transcend AUTO is the most innovative small, light and portable APAP, truly giving you the freedom to sleep anywhere.

Begin by inspecting all items shipped to you. Please report any damaged or missing items to the home healthcare provider where you obtained your device.

Note: For more detailed device descriptions, cautions and troubleshooting procedures refer to the Transcend AUTO User Manual that is available to download at www.mytranscend.com/support or by calling Somnetics customer service at 877-621-9626 to request a copy.

PRECAUTIONS WHEN USING A COMPUTER WITH TRANSCEND

The following precautions are cited for the safety of the patient and/or the person operating the computer as required to meet IEC 60601-1-1 safety regulations.

Definitions A computer compliant with 60950-1 safety standards is one that complies with UL 60950-1 or IEC 60950-1 safety standards.

- Do not plug any devices into the Transcend USB port other than a computer that is compliant with 60950-1 safety standards. Attaching any other device to the Transcend USB port may damage Transcend and may not be safe for the user.
- In order to reduce the risk of leakage currents, use an isolation transformer which is IEC 60601-1 approved to power your computer.
- Do not plug your computer compliant with 60950-1 safety standards or your Transcend into a multiple portable socket outlet (i.e. power strip).
- When using your computer compliant with 60950-1 safety standards, follow the manufacturer's cleaning instructions.
- When using your computer compliant with 60950-1 safety standards, follow the manufacturer's instructions for conducting preventative maintenance.
- Do not attach Transcend USB port to your computer compliant with 60950-1 safety standards during preventative maintenance of your computer.
- Do not touch your computer compliant with 60950-1 safety standards and any exposed metal on Transcend or on Transcend cables at the same time.
- Do not touch exposed metal on your computer compliant with 60950-1 safety standards or exposed metal on connectors or cables
- For clinicians, do not simultaneously touch the computer compliant with 60950-1 safety standards and the patient.
- Do not use computers that have internal voltages that are accessible without the use of tools in order to gain access to such voltages.

WARNINGS

- Do not allow water to enter this device. Transcend Auto should not be exposed to environmental conditions where the system may get wet.
- This device is not intended for life support.
- The Transcend Auto must be set up and adjusted by a trained provider before being used for therapy ramp and pressure.
- The air temperature produced by this device can be as much as 10°F higher than the temperature of the room. Exercise caution if the room temperature is warmer than 90°F (32°C).
- Do not block or otherwise obstruct the exhalation ports of the interface. Follow the manufacturer's instructions included with your interface.
- This equipment is not suitable for use with oxygen or in the presence of a flammable anesthetic mixture with air or oxygen, or with nitrous oxide.
- The Transcend Auto is only to be used with the supplied or recommended accessories. Use of accessories not recommended may result in increased electromagnetic emissions or decreased electromagnetic immunity of the PAP system and may be potentially unsafe.
- The Transcend Auto is not defibrillation proof.
- Do not attempt to sterilize Transcend Auto.
- If the device is to be used by multiple patients a main flow bacteria filter should be installed in-line between the device and the breathing circuit tubing to prevent contamination.
- The device should be used only with masks and connectors recommended by Somnedics or a health care professional. A mask should not be used unless the device is turned on and is properly delivering ramp or therapy pressure. The exhalation port(s) associated with the mask should never be blocked. Explanation of the Warning: The device is intended to be used with masks or connectors specifically designed to have exhalation ports to allow continuous flow of air out of the mask. When the device is turned on and functioning properly air flow from the device flushes the exhaled air out through the mask exhalation port. When the device is not operating, however, fresh air will not be provided through the mask and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- Unplug the Transcend Auto before cleaning.
- Do not submerge the Transcend Auto or power supply in liquid.
- Prevent water from entering any openings of the device.
- Do not use harsh or abrasive cleaning agents to clean the device or any components.
- Do not attempt to sterilize the Transcend Auto.
- Do not place cleaning materials, such as a cloth or liquid, into the device air inlet or air outlet connector.
- The air supply tubing, mask assembly, headgear, and bacterial/viral filters should be discarded after each patient use using standard institutional biohazard procedures. No attempt should be made to clean, disinfect, or sterilize these components for multiple users. These components are for single patient use. No attempt should be made to clean, disinfect or sterilize these components. Manufacturer's instructions must be followed with regard to cleaning, disinfecting, and re-use of patient interfaces (masks).

- Only use bacterial/viral filters that are commercially available and designed for use with CPAP machines. Follow the manufacturer's instructions for use of the filter. Only use those commercially available filters that do not require recalibration of the PAP device.
- Do not submerge the Transcend Auto or power supply in liquid. Do not allow liquid or cleaning solution to enter the device.
- Strangulation hazard from power cord and air tube. These can become wrapped around a neck and STRANGLE. Keep power cord and air tube more than 3 feet from a baby's crib and out of baby's reach. Keep cord and tube out of children's reach.
- Small parts are unlikely to be expelled from the Transcend enclosure, but in case of severe damage internal components may fragment and create a swallowing or choking hazard if they get out of the enclosure.

CAUTIONS

- Federal law (United States) restricts this device to sale by, or on the order of, a physician.
- Power the Transcend Auto only with the Somnetics-supplied power supplies, mobile power adapter, or batteries. See Appendix: Part Numbers.
- Discontinue use of the Transcend Auto and contact your physician if respiratory or skin irritations occur.
- Do not introduce objects into the Transcend Auto air inlet or air outlet.
- Inspect the power supply for signs of wear or damage before each use. Replace the power cord if necessary.
- Somnetics recommends replacing the air delivery tubing (hose) after every three months of use.
- To protect the environment, some parts and accessories of the Transcend Auto, including optional batteries, must be disposed of in accordance with local regulations.
- Transcend automatically converts voltage, but may require a plug adaptor for use in another country. It is recommended you use the interchangeable plug adaptors supplied with the Transcend power supply. Use only the Somnetics-supplied Universal Power Supplies and accessories.
- Do not use a power converter or voltage transformer with the PSA2 Universal Power Supply.

TRANSCEND COMPONENTS

Included with the Transcend AUTO:

- | | | | |
|---|--|---|----------------------------|
| 1 | Transcend AUTO | 5 | Changeable plug pack |
| 2 | Standard 6 foot (183 cm) hose – not HME compatible | 6 | Transcend Travel Bag |
| 3 | Universal hose adaptor | 7 | Transcend Auto Quick Guide |
| 4 | Multi-plug Universal AC Power Supply (PSA2) | 8 | USB cable |


What's Not Included (optional accessories sold separately):

- Transcend Heated Humidifier
- Transcend LCD Programming Base Station
- Transcend Base Station
- Transcend P₄ and P₈ Batteries
- Transcend Portable Solar Battery Charger
- Transcend Mobile Power Adaptor

ASSEMBLING YOUR TRANSCEND AUTO

To completely assemble the Transcend Auto:

1. Attach the Universal Hose Adaptor to the 6-foot air supply tube then connect the universal hose adaptor to the Transcend Auto making sure it is fully seated into the APAP.
2. Connect the interface (mask) to the patient (open) end of the 6-foot hose.
3. Plug the power supply barrel connector into the APAP with the cord pointed up.
4. Connect the power supply to a power source.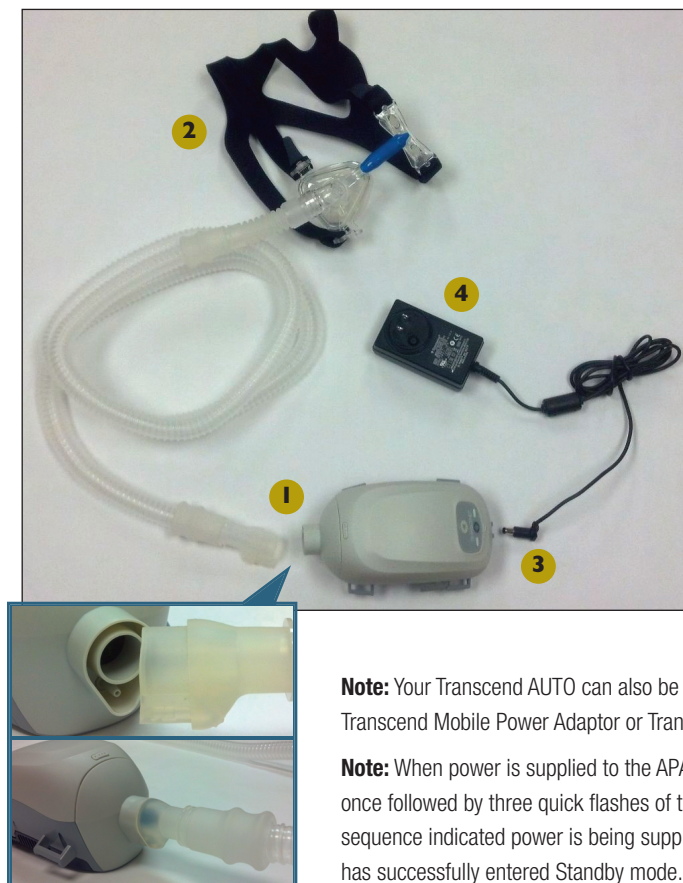

Note: Your Transcend AUTO can also be powered by the optional Transcend Mobile Power Adaptor or Transcend Battery.

Note: When power is supplied to the APAP, the yellow light will flash once followed by three quick flashes of the green LED light. This sequence indicated power is being supplied to the device and that it has successfully entered Standby mode.

Note: Do not use a power converter with your Transcend as it may damage the device. Transcend automatically converts voltage.

TO CHANGE THE PLUG ADAPTOR ON THE MULTI-PLUG UNIVERSAL POWER SUPPLY:


1. Depress the button on the power supply and rotate the plug adaptor counterclockwise to unlock.
2. Line up the new plug adaptor into the power supply and turn clockwise until the plug snaps into place.

STARTING AND ENDING THERAPY

Starting Therapy

1. The Transcend Auto control panel consists of a Power button (bottom), Ramp button (top), and two LED lights. (Refer to image)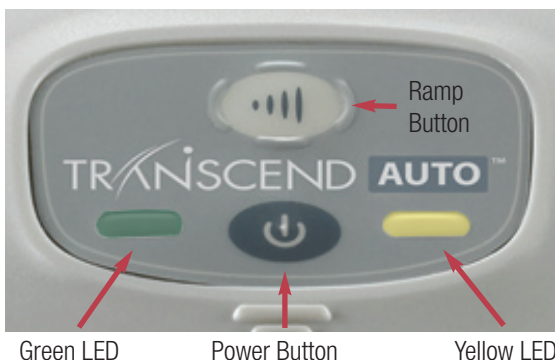

2. Ensure that the device is connected to a power source and is in Standby mode.
3. Put on your mask and make sure that proper fit is achieved.
4. Press the lower, gray Power button () to turn on Transcend Auto and enter Therapy mode. Air flow will begin as the blower delivers your prescribed therapy pressure.

Using the Ramp Function

Ramp settings will be programmed by your healthcare provider as prescribed.

1. Initiate ramp by pressing the Ramp (top) button after the Power button has been pressed.

Note: The Ramp button on the Transcend Auto will not work unless your healthcare provider has preset the ramp function.

2. Pressure from the device will be lowered to a preset starting pressure and will gradually increase over a preset time interval until the device reaches your prescribed starting therapy pressure.
3. To accelerate the ramp time, hold down the Ramp button. The device will quickly increase pressure until reaching your prescribed therapy pressure.

Ending Therapy

To end therapy, press the Power button to return the device to Standby mode.

CARING FOR YOUR TRANSCEND AUTO AND COMPONENTS

Drying Mode

Drying mode should be used daily to dry the inside of your Transcend Auto after use.

Caution: Do not attempt to use Transcend Auto while in Drying mode. The device will not provide prescribed therapy pressure while in Drying mode.

1. Set the device on a dry, stable surface. A cloth surface is not recommended as it may get soiled.
2. Make sure the device is in Standby mode.
3. To enable Drying mode, press and hold the Ramp button and then press the Power button.
4. Low pressure airflow will begin and last for 30 minutes until the drying sequence is completed. The device will then return to Standby mode. To end drying mode before the 30 minute sequence is completed, press the Power button. The device will return to Standby mode.

Caution: Drying mode will dry the inside of your Transcend Auto. Do not attempt to clean the inside of the Transcend device.

Cleaning the Exterior

Clean the exterior of your Transcend Auto device as needed. To clean, wipe the surface with a lint-free cloth or paper towel that has been dampened in water containing mild dishwashing detergent*. Following cleaning, wipe again with a cloth or paper towel moistened with clear water. Allow to dry.

Caution: Never submerge Transcend Auto in water. Do not allow soap or water to enter the device.

Cleaning the Transcend Auto Components

The interface (mask) should be washed weekly. The air supply tube (hose) should be washed daily.

1. Completely disconnect the components.
2. Hand wash in warm water with a mild dishwashing detergent*.
3. Rinse thoroughly.
4. Allow to air dry. Components should be completely dry before reconnecting.
5. Reconnect the components and reattach to Transcend.



Caution: Do not put any Transcend Auto components in the dishwasher or laundry machine.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

CLEANING AND REPLACING THE FILTER MEDIA

The Transcend AUTO Filter Media should be replaced every six months. Contact your homecare dealer to purchase a replacement Filter Media.

1. Remove the filter frame by gently pressing the button release and lifting the plate away from the APAP (Image 1).
2. Remove the Filter Media from the filter frame and discard.
3. Insert a new Filter Media into the filter frame (Image 2).
4. To replace the new filter frame, lay it on the bottom of the APAP. Snap the front clips into place first, then snap the connector into place.

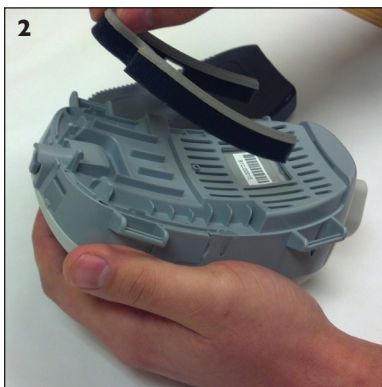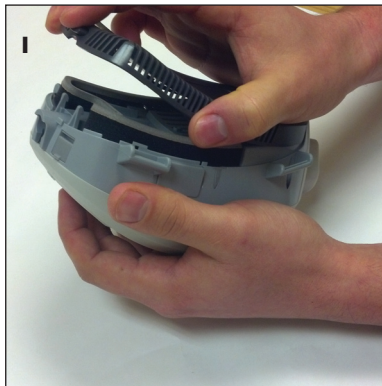The Filter Media should be removed and cleaned weekly.

1. To clean the Filter Media, first submerge a lint-free cotton cloth into the cleaning solution*.
2. Wring excess water from the cloth.
3. Gently pat the Filter Media with the cloth.
4. Dry surfaces by wiping with a dry, lint-free cotton cloth.
5. Allow the Filter Media to air dry at room temperature for approximately 30-60 minutes before reassembly. Drying time will depend on ambient conditions.
6. Reconnect the Filter Media and filter frame to the Transcend AUTO.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

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©2017 Somnetics International, Inc. All rights reserved.
ATLANTICO SYSTEMS LTD.