Transcend® Quick Guide

To view Transcend Instructional videos
go to www.mytranscend.com/support
or scan with QR code reader:

ENGLISH

FRANÇAIS

DEUTSCH

ITALIANO

ESPAÑOL

NEDERLANDS

PORTUGUÊS

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TRANSCEND INTRODUCTION

Thank you for choosing Transcend, the first obstructive sleep apnea therapy device designed to fit your active lifestyle. Weighing less than a pound (426 g) Transcend is the most innovative small, light and portable CPAP, truly giving you the freedom to sleep anywhere.

R_x only

Begin by inspecting all items shipped to you. Please report any damaged or missing items to the home healthcare provider where you obtained your device.

Note: For more detailed device descriptions, cautions and troubleshooting procedures refer to the Transcend Sleep Apnea Therapy System User Manual that is available to download at www.mytranscend.com/support or by calling Somnetics customer service at 877-621-9626 to request a copy.

PRECAUTIONS WHEN USING A COMPUTER WITH TRANSCEND

The following precautions are cited for the safety of the patient and/or the person operating the computer as required to meet IEC 60601-1-1 safety regulations.

Definitions A computer compliant with 60950-1 safety standards is one that complies with UL 60950-1 or IEC 60950-1 safety standards.

- Do not plug any devices into the Transcend USB port other than a computer that is compliant with 60950-1 safety standards. Attaching any other device to the Transcend USB port may damage Transcend and may not be safe for the user.
- In order to reduce the risk of leakage currents, use an isolation transformer which is IEC 60601-1 approved to power your computer.
- Do not plug your computer compliant with 60950-1 safety standards or your Transcend into a multiple portable socket outlet (i.e. power strip).
- When using your computer compliant with 60950-1 safety standards, follow the manufacturer's cleaning instructions.
- When using your computer compliant with 60950-1 safety standards, follow the manufacturer's instructions for conducting preventative maintenance.
- Do not attach Transcend USB port to your computer compliant with 60950-1 safety standards during preventative maintenance of your computer.
- Do not touch your computer compliant with 60950-1 safety standards and any exposed metal on Transcend or on Transcend cables at the same time.
- Do not touch exposed metal on your computer compliant with 60950-1 safety standards or exposed metal on connectors or cables
- For clinicians, do not simultaneously touch the computer compliant with 60950-1 safety standards and the patient.
- Do not use computers that have internal voltages that are accessible without the use of tools in order to gain access to such voltages.

WARNINGS

- Do not allow water to enter this device. Transcend should not be exposed to environmental conditions where the system may get wet.
- This device is not intended for life support.
- The Transcend must be set up and adjusted by a trained provider before being used for therapy ramp and pressure.
- The air temperature produced by this device can be as much as 10°F higher than the temperature of the room. Exercise caution if the room temperature is warmer than 90°F (32°C).
- Do not block or otherwise obstruct the exhalation ports of the interface. Follow the manufacturer's instructions included with your interface.
- This equipment is not suitable for use with oxygen or in the presence of a flammable anesthetic mixture with air or oxygen, or with nitrous oxide.
- The Transcend is only to be used with the supplied or recommended accessories. Use of accessories not recommended may result in increased electromagnetic emissions or decreased electromagnetic immunity of the PAP system and may be potentially unsafe.
- The Transcend is not defibrillation proof.
- Do not attempt to sterilize Transcend.
- The device should be used only with masks and connectors recommended by Somnedics or a health care professional. A mask should not be used unless the device is turned on and is properly delivering ramp or therapy pressure. The exhalation port(s) associated with the mask should never be blocked. Explanation of the Warning: The device is intended to be used with masks or connectors specifically designed to have exhalation ports to allow continuous flow of air out of the mask. When the device is turned on and functioning properly air flow from the device flushes the exhaled air out through the mask exhalation port. When the device is not operating, however, fresh air will not be provided through the mask and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- Unplug the Transcend before cleaning.
- Do not submerge the Transcend or power supply in liquid.
- Prevent water from entering any openings of the device.
- Do not use harsh or abrasive cleaning agents to clean the device or any components.
- Do not attempt to sterilize the Transcend.
- The system is intended for single patient use.
- Do not place cleaning materials, such as a cloth or liquid, into the device air inlet or air outlet connector.

- Strangulation hazard from power cord and air tube. These can become wrapped around a neck and STRANGLE. Keep power cord and air tube more than 3 feet from a baby's crib and out of baby's reach. Keep cord and tube out of children's reach.
- Small parts are unlikely to be expelled from the Transcend enclosure, but in case of severe damage internal components may fragment and create a swallowing or choking hazard if they get out of the enclosure.

CAUTIONS

- Federal law (United States) restricts this device to sale by, or on the order of, a physician.
- Power the Transcend only with the Somnetics-supplied power supplies, mobile power adapter, or batteries. See Appendix: Part Numbers.
- Discontinue use of the Transcend and contact your physician if respiratory or skin irritations occur.
- Do not introduce objects into the Transcend air inlet or air outlet.
- Inspect the power supply for signs of wear or damage before each use. Replace the power cord if necessary.
- Somnetics recommends replacing the air delivery tubing (hose) after every three months of use.
- To protect the environment, some parts and accessories of the Transcend, including optional batteries, must be disposed of in accordance with local regulations.
- Transcend automatically converts voltage, but may require a plug adaptor for use in another country. It is recommended you use the interchangeable plug adaptors supplied with the Transcend power supply. Use only the Somnetics-supplied Universal Power Supplies and accessories.
- Do not use a power converter or voltage transformer with the PSA2 Universal Power Supply.

TRANSCEND COMPONENTS

Included with the Transcend CPAP:

- | | | | |
|---|--|---|-----------------------|
| 1 | Transcend CPAP | 5 | Changeable Plug Pack |
| 2 | Standard 6 foot (183 cm) hose – (Note: not compatible with optional H6B or H9M Waterless Humidification Systems) | 6 | Transcend Travel Bag |
| 3 | Universal hose adaptor | 7 | Transcend Quick Guide |
| 4 | Multi-plug Universal AC Power Supply (PSA2) | 8 | USB cable |


What's Not Included (optional accessories sold separately):

- | | |
|--|---|
| • Transcend Heated Humidifier | • Transcend P ₄ and P ₈ Batteries |
| • Transcend LCD Programming Base Station | • Transcend Portable Solar Battery Charger |
| • Transcend Base Station | • Transcend Mobile Power Adaptor |

ASSEMBLING YOUR TRANSCEND

To completely assemble the Transcend Sleep Apnea Therapy System:

1. Attach the Universal Hose Adaptor to the 6-foot air supply tube then connect the universal hose adaptor to the Transcend making sure it is fully seated into the CPAP.
2. Connect the interface (mask) to the patient (open) end of the 6-foot hose.
3. Plug the power supply barrel connector into the CPAP with the cord pointed up.
4. Connect the power supply to a power source.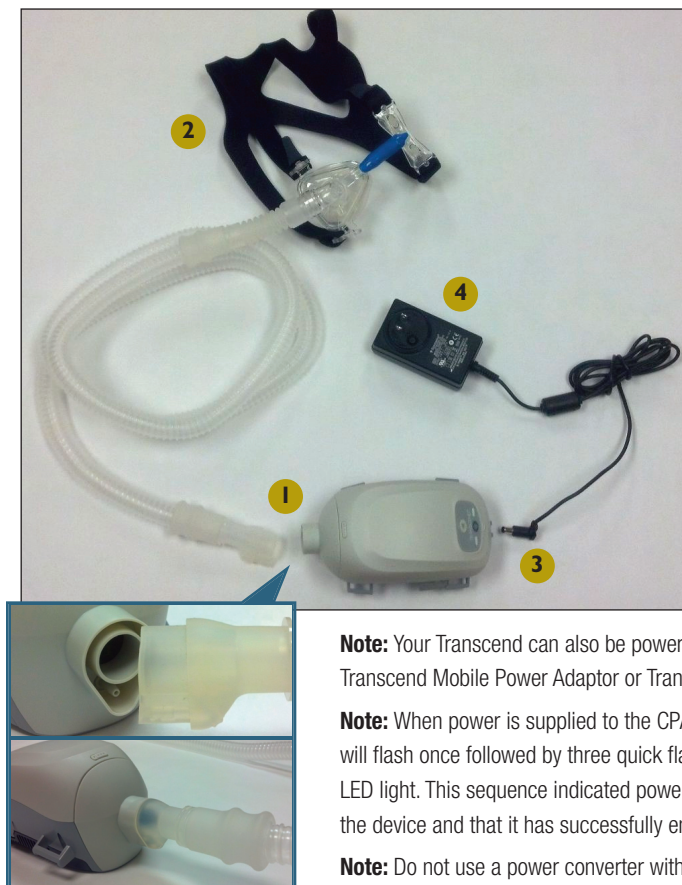

Note: Your Transcend can also be powered by the optional Transcend Mobile Power Adaptor or Transcend Battery.

Note: When power is supplied to the CPAP, the yellow light will flash once followed by three quick flashes of the green LED light. This sequence indicated power is being supplied to the device and that it has successfully entered Standby mode.

Note: Do not use a power converter with your Transcend as it may damage the device. Transcend automatically converts voltage.

TO CHANGE THE PLUG ADAPTOR ON THE MULTI-PLUG UNIVERSAL POWER SUPPLY:


1. Depress the button on the power supply and rotate the plug adaptor counterclockwise to unlock.
2. Line up the new plug adaptor into the power supply and turn clockwise until the plug snaps into place.

STARTING AND ENDING THERAPY

Starting Therapy

1. The Transcend control panel consists of a Power button (bottom), Ramp button (top), and two LED lights. (Refer to image)
2. Ensure that the device is connected to a power source and is in Standby mode.
3. Put on your mask and make sure that proper fit is achieved.
4. Press the lower, gray Power button () to turn on Transcend and enter Therapy mode. Air flow will begin as the blower delivers your prescribed therapy pressure.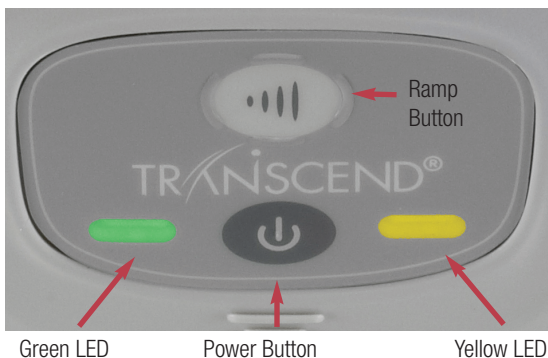

Using the Ramp Function

Ramp settings will be programmed by your healthcare provider as prescribed.

1. Initiate ramp by pressing the Ramp (top) button after the Power button has been pressed.

Note: The Ramp button on the Transcend will not work unless your healthcare provider has preset the ramp function.

2. Pressure from the device will be lowered to a preset starting pressure and will gradually increase over a preset time interval until the device reaches your prescribed therapy pressure.
3. To accelerate the ramp time, hold down the Ramp button. The device will quickly increase pressure until reaching your prescribed therapy pressure.

Ending Therapy

To end therapy, press the Power button to return the device to Standby mode.

CARING FOR YOUR TRANSCEND AND COMPONENTS

Drying Mode

Drying mode should be used daily to dry the inside of your Transcend after use.

Caution: Do not attempt to use Transcend while in Drying mode. The device will not provide prescribed therapy pressure while in Drying mode.

1. Set the device on a dry, stable surface. A cloth surface is not recommended as it may get soiled.
2. Make sure the device is in Standby mode.
3. To enable Drying mode, press and hold the Ramp button and then press the Power button.
4. Low pressure airflow will begin and last for 30 minutes until the drying sequence is completed. The device will then return to Standby mode. To end drying mode before the 30 minute sequence is completed, press the Power button. The device will return to Standby mode.

Caution: Drying mode will dry the inside of your Transcend. Do not attempt to clean the inside of the Transcend device.

Cleaning the Exterior

Clean the exterior of your Transcend device as needed. To clean, wipe the surface with a lint-free cloth or paper towel that has been dampened in water containing mild dishwashing detergent*. Following cleaning, wipe again with a cloth or paper towel moistened with clear water. Allow to dry.

Caution: Never submerge Transcend in water. Do not allow soap or water to enter the device.

Cleaning the Transcend Components

The interface (mask) should be washed weekly. The air supply tube (hose) should be washed daily.

1. Completely disconnect the components.
2. Hand wash in warm water with a mild dishwashing detergent*.
3. Rinse thoroughly.
4. Allow to air dry. Components should be completely dry before reconnecting.
5. Reconnect the components and reattach to Transcend.

Caution: Do not put any Transcend components in the dishwasher or laundry machine.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

CLEANING AND REPLACING THE FILTER MEDIA



The Transcend Filter Media should be replaced every six months. Contact your homecare dealer to purchase a replacement Filter Media.

1. Remove the filter frame by gently pressing the button release and lifting the plate away from the CPAP (Image 1).
2. Remove the Filter Media from the filter frame and discard.
3. Insert a new Filter Media into the filter frame (Image 2).
4. To replace the new filter frame, lay it on the bottom of the CPAP. Snap the front clips into place first, then snap the connector into place.

The Filter Media should be removed and cleaned weekly.

1. To clean the Filter Media, first submerge a lint-free cotton cloth into the cleaning solution*.
2. Wring excess water from the cloth.
3. Gently pat the Filter Media with the cloth.
4. Dry surfaces by wiping with a dry, lint-free cotton cloth.
5. Allow the Filter Media to air dry at room temperature for approximately 30-60 minutes before reassembly. Drying time will depend on ambient conditions.
6. Reconnect the Filter Media and filter frame to the Transcend.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

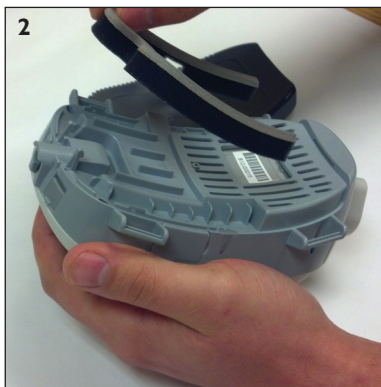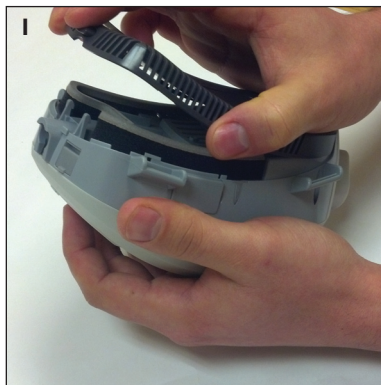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ATLANTICO SYSTEMS LTD.